

CenPACS 6.6.2 Quick Guide – Study Browser

Displays the filter for customizing the studies shown - See below

Selects all studies on this page

Double-click the "VIEWS" menu to quickly return to the EasyWEB page

Offers additional methods for searching - by Patient, Study, Series, Image, or Worklist data

Help Manual
Contains User and Admin Guides for help on features

Contains Profile settings for customizing info displayed

Click the eyeball to launch iQ-X, iQ-4VIEW or CenPACS Viewer.

Begins the Search process

Click the "+" to drill into the series or image levels

Click the box to select the study

Offers additional views for studies - "EasyWEB" is the most common

Search
Enter a combination of criteria in the fields below the table headers to search for patients or studies. The more criteria entered, the more narrow the search will be. Wildcard characters "*" and "?" can be used for less specific matching.

	PATIENT ID	PATIENT NAME	SEX	DATE OF BIRTH	AGE	INSTITUTION NAME
<input type="checkbox"/>	MR6592	PATIENT6592	M	1901-01-01 [113]	53	GENERIC INSTITUTION
<input type="checkbox"/>	MR6486	PATIENT6486	F	1901-01-01 [113]	64	GENERIC INSTITUTION
<input type="checkbox"/>	PATID3	TEST, PDF	O			

Filter The "Show Filter" button above displays the Filter area where each user can customize their view. All criteria must be met for a study to display.

Adds a quick link icon to Study Notes for the study

Opens iQ-X or iQ-4VIEW in the current window/tab or a new one

Determines if the Filters area closes when "Apply Filter" is clicked

Refreshes the EasyWEB page based on the value

Clears any Filter criteria

of studies to display at a time

Display read or unread studies

Set dates or date ranges to display

Select specific modalities

Choose specific physicians

"Date Received" is when the study was received into the PACS - NOT the Study Date

Once selections have been made, click here to apply them

Choose specific physicians

"Date Received" is when the study was received into the PACS - NOT the Study Date

Once selections have been made, click here to apply them

